


# medistus®

## ANTIVIRUS

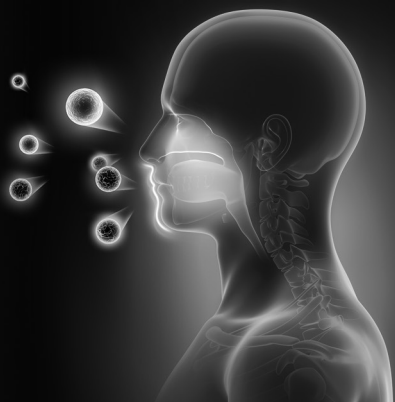

travel public transportation education waiting rooms shopping event



MEDICAL DEVICE CE 0481

## INSTRUCTIONS FOR USE

Please read the instructions for use carefully before taking Medistus® Antivirus lozenges.

### What are Medistus® Antivirus lozenges?

Medistus® Antivirus lozenges contain Kistosyn® 200 extract.

Medistus® Antivirus is a medical device product. Efficacy and safety have been tested according to the European Medical Device Directive (93/42/EEC).

### Main effect of Medistus® Antivirus lozenges

By purchasing Medistus® Antivirus you have acquired a medical device with an active ingredient complex containing plant-based substances. The Kistosyn® 200 extract forms through its natural active substances a protective film over the oral and pharyngeal mucosa. This protective barrier leads to a physical (mechanical) protection against

viruses and bacteria causing respiratory infections, which helps prevent their penetration into body cells and their further propagation.

The Kistosyn® 200 extract supports through its natural active substances the barrier function of the oral and pharyngeal mucosa by forming a natural film on it. As a result, external irritations can significantly be reduced. In addition, the Kistosyn® 200 extract supports the protection of the oral mucosa from inflammations and thereby the alleviation of cough and dry cough.

Due to this physical effect resistance development is not possible.

### Areas of application


In which cases shall Medistus® Antivirus be applied?

- To support by physical means the natural barrier function of the oral and pharyngeal mucosa against viruses and bacteria infecting the respiratory tract.
- For the prevention of infectious and inflammatory airways diseases of viral and bacterial origin by physical (mechanical) barrier formation on the pharyngeal mucosa.
- For the alleviation of cough and dry cough.
- Also applicable at the first signs of scratching in the throat.

### Dosage instructions, type and duration of application

When, how often and how long shall Medistus® Antivirus be applied?

- For protection at increased risk of airways infection of viral and bacterial origin.
- Even at the first signs of scratching in the throat.
- Adults: Dissolve a lozenge slowly in the mouth 3 to 6 times a day.

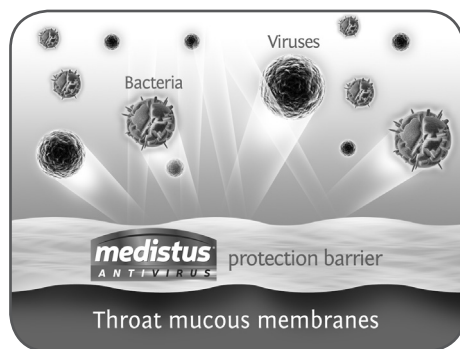

# medistus®

## A N T I V I R U S



travel



public transportation



education




waiting rooms



shopping



event

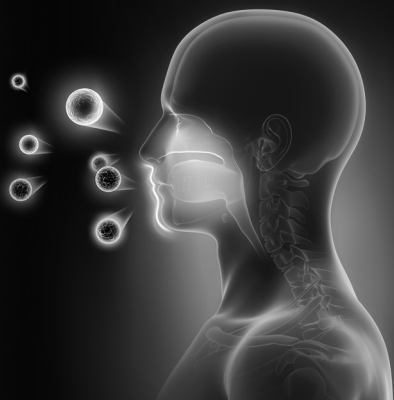

MEDICAL DEVICE  0481

- Children older than 12 years (and of at least 25 kg body weight): Dissolve a lozenge slowly in the mouth 3 times a day.
- Children older than 6 years: Dissolve a lozenge slowly in the mouth once a day.

The lozenges may pose a choking risk for small children when unable to dissolve them in the mouth. Please take particular care in case of young children. In case of taking medicines orally, please leave a pause of 4 hours in-between.

If within 2-3 days after the initiation of the treatment no improvement occurs or the symptoms aggravate (such as upon the occurrence of fever over 38°C), please consult your doctor.

### Please note

Keep out of reach of small children. Suitable for children older than 6 years.  
Suitable also for pregnant and breastfeeding women.

### Side effects

There are no side effects known.

### Counter indications

There are no counter indications known.

### Interactions

To avoid interactions, in case of taking medicines orally, please leave a pause of 4 hours between the intakes or consult your doctor.

### Notes and information concerning the shelf life of the medical device

Medistus® Antivirus lozenges shall be used up until the „best before“ date marked on the box.

### Form and content

Original packaging with 10 lozenges.

### Ingredients

Active ingredients: Kistosyn® 200 extract  
Other ingredients: maltitol, sorbitol, purified water, ascorbic acid, citric acid, flavor, vegetable oil, stevia extract, peppermint oil.  
No preservatives and colorants.  
Sugar, gluten and lactose free.

### Storage

Store in a cool, dry place, protected from light.


### Disposal

The product can be disposed of with the regular household waste.

### Medistus® Antivirus lozenges

- **act as a protective film**
- **form a natural barrier**
- **assist preventively and in the alleviation of infection**
- **alleviate cough and dry cough**

### Natural health protection at increased infection risk

 **Manufacturer:** Nutrin GmbH  
Schnaid 36b, 91352 Hallerndorf / Germany

**License owner:** Innopharm GmbH  
Hauptplatz 1, 1. OG, A-7350 Oberpullendorf / Austria

**Distributor:** VitaPlus Ltd.  
1037 Budapest, Csillaghegyi út 19-21.  
[www.medistusantivirus.com](http://www.medistusantivirus.com)

The Medistus figurative trademark and the Kistosyn word trademark are registered international trademarks of VitaPlus Ltd.

MATGB102\_I1